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**Safety goggle is non-sterile disposable anti-fog goggle.**

1. Brand: Refine

2. Product name: Safety goggle

3. Material: PVC 70% PC 30%

4. Model: EP01, EP02, EP05

5. Composition: composed of protective cover and fixing device made of polymer material. Non-sterile, single use.

6. Product Standards: GB 14866-2006 / EN 166: 2001 / ANSI / ISEA Z87.1-2015  
AS / NZS 1337.1: 2010 / CSA Z94.3-2015

7. Intended use: Safety Goggle is common eye protection equipment, intended to protect against mechanical impact, mechanical injuries, liquids, ocular contact.

Recommended industry: Facility Sanitation, Food Processing, Food Safety, which are used for medical staffs use, to prevent crossing infection between medical staffs and patients and to protect patients.

8. Product instruction: The product and the protective packaging should be checked before using. Stop using it if there is any damage.

9. Precautions:

- One-time use only.
- Use in accordance with the instructions.
- Never use it when damages are found.
- Stay away from chemicals.

10. Storage

Please store in clean, dry and ventilated indoor place, with relative humidity (10%-93%), Atmospheric pressure for storage (70kPa-106kPa), temperature (-20°C- +40 °C), avoid corrosive gas.

Warning:

- Materials which may come into contact with the wearer's skin could cause allergic reactions to susceptible individuals;
- Scratched or damaged oculars should be replaced;
- eye-protectors against high speed particles worn over standard ophthalmic spectacles may transmit impacts, thus creating a hazard to the wearer.
- These protectors are intended for indoor use where no optical radiation hazards exist.

 Guilin Refine Medical Instrument Co., Ltd.

No.8-3, Information Industrial Park, High-Tech Zone, Qixing District, Guilin, Guangxi, 541004, P.R. China

Tel: +86-773-7796686 E-mail: refine@refine-med.com

http://www.refine-med.com Shelf life:2 years.

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MedPath GmbH (PPE)
MedNet EC-REP GmbH (Medical)